
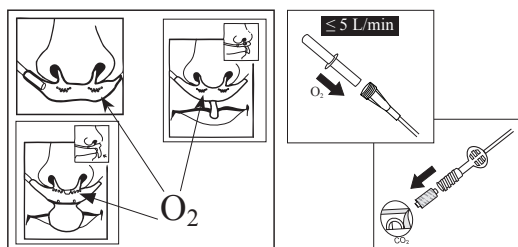
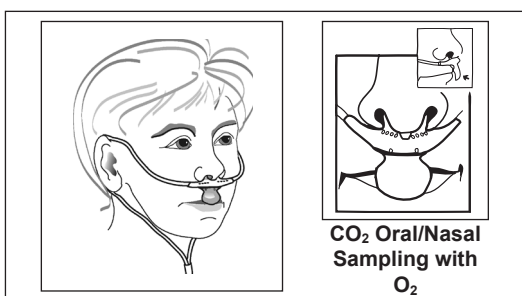
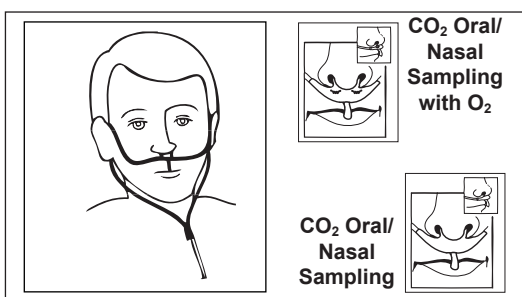
Omnistream™

ETCO₂ SAMPLING

Products for Non-Intubated Patients Directions for Use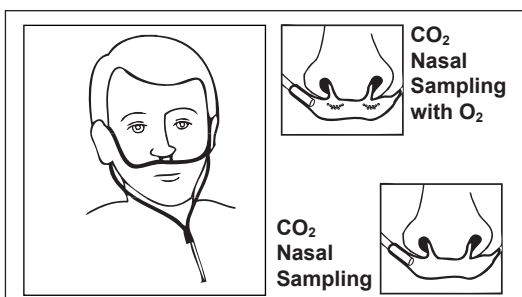

© 2010 Oridion Medical 1987 Ltd.
All rights reserved.
PN 007637H

Oridion®

ENG

Omnistream™ EtCO₂ Sampling for Non-Intubated Patients

Single patient use, not to be re-processed
Non-sterile / non-pyrogenic
Latex free

- Warning:** Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.
- Warning:** Do not cut or remove any part of the sample line. Cutting the sample line could lead to erroneous readings.
- Caution:** Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.
- Caution:** Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.
- Note:** Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

	Do not reuse
	Latex Free
	Fire Hazard
	Storage temperature
	Rx Only
	Caution, consult accompanying documents
	CE Mark

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Obelis S.A.
Oridion Capnography Inc.